Pro-ophta® Junior

Okklusionspflaster
Pansement occlusif
Occlusive Dressing
Parche ocular
Penso oclusivo
Cerotto occlusivo
Occlusiepleister
Okklusionsplaster
Ockklusionsplåster
Okluzní náplast
Oklúzna náplast
Oklúziós tapasz
Plaster okluzyjny

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en Instructions for Use

Product description and performance characteristics

The Pro-ophta® Junior occlusive dressings are used as an opaque covering for the eye during occlusion therapy for strabism and amblyopia. The thin, supple material of the Pro-ophta® Junior occlusive dressings is suitable for sensitive skin and are easy to remove, leaving little residue.

Product composition

Nonwoven base material: 100% polyester
Opaque centre: 100% cotton fabric

Intended purpose

The Pro-ophta® Junior occlusive dressings are used for covering the eye during occlusion therapy.

Indications

- Ambylopia
- Strabism

Contraindications

Known allergy and/or hypersensitivity to any of the product components.

Side effects

In very rare cases skin irritations and/or allergies may occur.

Warnings and precautions

None

Usage information

Ensure before applying the occlusive dressing that the patient's skin is clean, dry and free of any creams and ointments and that any wounds are covered.

1. Application

- Tear the cover paper along the perforation provided for this purpose. Remove the cover paper from the dressing.
- Apply the dressing with the longest edge over the closed eye.
- Place the opaque grey area directly in front of the eye, ensuring the eyelid can still move freely.
- Firmly press the dressing onto the skin around the eye.
- If desired, you can now decorate the occlusive dressing with the enclosed stickers.

2. Dressing change

The treating doctor will determine the periods for which the dressing must be worn. This depends on the medical findings and the patient's treatment plan.

3. Disposal:

The product contains zinc oxide, which is classed as chronically hazardous to the aquatic environment (toxic for aquatic organisms). There are no safety concerns regarding skin contact with zinc oxide. In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance - AVV). Recyclable packaging should be recycled according to the relevant national recycling systems.

General instructions:

The Pro-ophta® Junior occlusive dressings are intended for single use and must not be re-used.
Contains zinc oxide, please take note of the specific information in the "Disposal" section.

In case of a serious incident, contact the manufacturer and the responsible health authorities.

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