Cellacare® Rhizocast Classic

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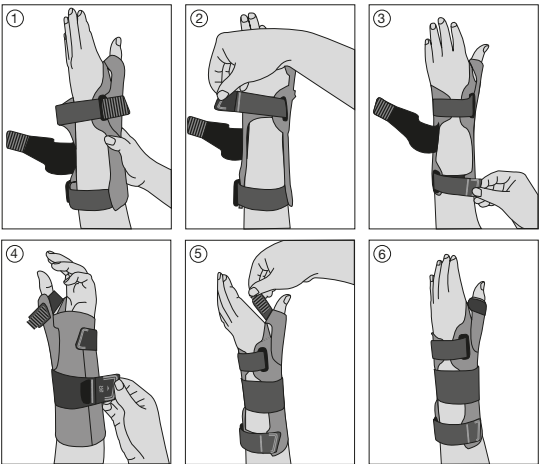
Wrist and thumb orthosis

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Ortesi del polso e del pollice

Pols- en duimorthese

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en Instructions for use

Dear customer,

Thank you for choosing an L&R product.

Please read the instructions for use carefully. If you have any questions, please contact your physician or specialist shop.

Product description and performance characteristics:

Cellacare® Rhizocast Classic is an orthosis for the wrist and the thumb. The combination of one palmar stay, one ulnar stay, two radial stays which also include the thumb and four stabilising straps immobilise the wrist, the carpometacarpal (saddle or basal) joint and the metacarpophalangeal joint of the thumb. The stays can be adjusted individually by a specialist if necessary.

Product composition:

polyamide, polyester, polyurethane, cotton, polyacetal, aluminium

Intended purpose:

Orthosis for immobilisation of the wrist and thumb

Indications:

- immobilisation in cases of posttraumatic, postoperative conditions in the wrist area and the articulatio metacarpophalangealis I (thumb basal joint) and / or articulatio carpometacarpalis pollicis (saddle joint of the thumb) area and in the case of ulnar ligament injury (skier's thumb)
- immobilisation in case of degenerative changes in the wrist area and the articulatio metacarpophalangealis I (thumb basal joint) and / or articulatio carpometacarpalis pollicis (saddle joint of the thumb) area (scaphotrapezio-trapezoid arthritis, rhizarthritis)
- immobilisation in the case of inflammation of the tendons of the hand (tendovaginitis)
- immobilisation in the case of initial carpal tunnel syndrome (CTS)

Contra-indications:

- known allergy and / or hypersensitivity to any of the product components

Side effects:

- In very rare cases, skin irritations and / or allergies may occur.

Warnings and precautions:

- Do not use Cellacare® Rhizocast Classic unless directed by a health care professional. The first time patients use the product they need to have a health care professional show them how to put it on correctly.
- The health care professional will determine how long the product is to be worn per day, based on the indications and healing progress.
- Cellacare® Rhizocast Classic shall only be used on intact skin. Wounds have to be covered with a suitable wound dressing before use.
- In case of skin diseases and / or injuries in the application area (open, non-intact or injured skin) please contact your treating physician / seek medical advice before application of the product.
- In case of specific morphological peculiarities in the wrist or thumb being treated which restrict or complicate treatment with the orthosis and its correct application, please contact your treating physician / seek medical advice before application of the product.
- Apply the product in accordance with the instructions for use and illustrations provided. Please check the positioning of the product during use and correct as necessary.
- In case of discomfort or problems, consult a health care professional.
- If local swelling or numbness occurs in the part of the body being treated, the product must be removed and a medically qualified health care professional consulted.

- Cellacare® Rhizocast Classic is intended for use on a single patient only.
- The product's durability may be compromised by certain factors, e.g. objects with sharp edges or damage to the hook-and-loop fastener. To prevent this, the hook-and-loop fastener should always be fastened, for example when the product is not being worn or when being washed.
- Cellacare® Rhizocast Classic must not be used beyond the last day of the month indicated on the packaging (expiry date).
- If there are excessive signs of wear that could influence the performance of the product, consult a healthcare professional and replace the product if necessary.

Usage information:

Usage:

1. Open all four hook-and-loop fasteners. Make sure the straps do not slip out of the buckles. Then slide your hand and thumb into the orthosis so that your thumb goes into the appropriate hole (Fig. 1).
2. Close the strap that goes across your hand by pulling it further through the buckle, and fasten it to the orthosis in the opposite direction using the hook-and-loop fastener (Fig. 2). Repeat this process with the strap that goes across the forearm (Fig. 3).
3. Guide the wide strap around the wrist and affix it using the hook-and-loop fastener (Fig. 4).
4. Pull the small strap around the thumb and affix it using the hook-and-loop-fastener (Fig. 5).
5. Check the fit of the orthosis and the straps (Fig. 6) and if necessary adjust the pressure level by adjusting the tension of the straps. It is recommended that the stays be individually adapted by a health care professional to obtain an optimal fit of the orthosis.

Care recommendations:

Wash the orthosis at 30 °C (delicates cycle) with like colours. Do not bleach. Allow Cellacare® Rhizocast Classic to air-dry away from heat sources; do not dry in a dryer. Do not iron and do not dry clean. Fasten the hook-and-loop closures regularly so they retain their ability to fasten tightly, and to preserve the performance of the product.

Storage and transport conditions:

Cellacare® Rhizocast Classic must be kept away from light and moisture (preferably in the original packaging).

Disposal:

In Europe, product waste can be assigned a waste code from Chapter 18 01 and packaging waste a waste code from Chapter 15 01 of the Directive on the European Waste Catalogue (Waste Catalogue Ordinance – AVV). Packaging labelled as recyclable should be taken to the applicable national recycling systems.

General information:

In case of a serious incident, contact the manufacturer and the responsible health authorities.

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