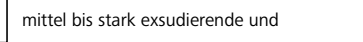
Suprasorb® P sensitive border

Silikon-Schaumverband
Pansement hydrocellulaire siliconé
Silicone Foam Dressing
Apósito de espuma y silicona
Ligadura esponjosa de silicone
Medicazione in schiuma di silicone

Siliconen schuimverband
Silikone-schuimverband
Silikonskumförband
Silikon-skumbandasje
Silikonivaahotosidos
Silikonový pěnový obvaz
Silikónový penový obväz
Szilikon habkötés
Opatrunek z pianki z warstwą silikonu
Obloga iz silikonske pene
Povoj sa silikonskom pjenošom
Απώδεος σιλικονούχο επίθεμα
Silikonlu köpük yara örtüsü
ضمادة السيليكون الإسفنجية

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tiefe* Wunden:

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*Where necessary, for deep wounds apply a rope as primary dressing (e.g. Suprasorb® A, Suprasorb® A+Ag, Suprasorb® Liquecel).
3. Dressing change
The dressing can remain on the wound for several days, dependant on wound condition and the level of exudate.

Precautions and warnings

- The product is for single use only.
- Do not use it if the packaging is damaged or opened.
- Do not use Suprasorb® P sensitive Silicone Foam Dressing together with oxidising agents such as hypochlorite solutions or hydrogen peroxide.
- If reddening or sensitisation occur discontinue use and consult a health care professional for appropriate treatment.
- Because of the lack of data, cautious use is advised in sensitive population groups such as infants, children, pregnant or nursing women.

Note

In case of serious incident, contact the manufacturer and the responsible health authorities.

Storage and disposal

1. Keep the product in its original packaging in a cool, dry place in specified temperature range and away from sunlight.
2. Disposal should be handled according to local environmental procedures.

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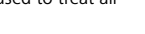
Product composition

Silicone, polyurethane, polyacrylate, fluff pulp and polyolefin

Intended purpose

Suprasorb® P sensitive Silicone Foam Dressing may be used to treat all wounds, provided they are

moderately to heavily exuding and

superficial to

deep* :

Indications

- Pressure ulcers
- Leg and foot ulcers
- Traumatic wounds
- Surgical wounds
- Skin tears

The dressing may also be used as part of a prophylactic therapy to help prevent pressure ulcers.

Contraindications

Known allergy and/or hypersensitivity to any of the product components

Application instructions

The product can be used under compression bandaging.

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- c. Remove the remaining release liner(s) and smooth down the border on the skin. Do not stretch the dressing.

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